


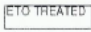

















INSTRUCTIONS FOR USE

ScopeFlow Disposable Air Water and Suction Button Set

The buttons are designed to be compatible with the Olympus 140 and 240 series flexible endoscopes and above. The disposable buttons are intended to replace the re-usable Air Water and Suction button with a single patient use alternative.

SYMBOL LEGEND :

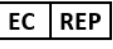								
Manufacturer	Do not use if pack is open or damaged	Consult Instructions for Use	Ethylene oxide treated	Single use	Date of manufacture	Use by date	Product reference number	Lot number
								
Keep away from sunlight	Keep dry	Non sterile	CE Mark	Do not re-sterilize	Not made with Natural rubber latex	Not made with DEHP	Biohazard	Medical device
								
EC Rep								

PRODUCT CODES AND DESCRIPTIONS

Product code	Description	Endoscope compatibility (Gastroscope – Coloscope)
PFE306	ScopeFlow Air Water and Suction button set	Olympus® series 140/ 240, 160/260, 180/280, 190/290

INTENDED USE

The ScopeFlow Air Water and Suctions buttons are designed to be used with an Olympus endoscope during a procedure to allow air or water to pass through and control the suction of bodily fluids through the working channel. They are designed to replace the reusable Olympus buttons which are historically difficult to reprocess.

INSTRUCTIONS FOR USE

Consult the instructions for use of the endoscope as well as the concerned accessories to be before using this medical device.

This medical device must only be used by trained and qualified medical staff or authorized person familiar with endoscopy techniques and acquainted with the pathologies in question as well as their possible complications.

DIRECTIONS FOR USE

- 1) Ensure packaging is not damaged or opened before use.
- 2) Remove Air/Water and Suction buttons from the package.

- 3) Attach Suction button to the suction port on the head of the flexible endoscope.
- 4) Take the Air/Water Button and moisten the black seals with sterile water.
- 5) Attach the Air/Water Button to the port on the head of the flexible endoscope.
- 6) Before the procedure, test the suction of the endoscope by pressing the suction button and ensuring suction is adequate.
- 7) Check the Air/Water button provides sufficient air and water.
- 8) Discard the buttons after the procedure has finished.

WARNINGS AND PRECAUTIONS

The manufacturer did not design this device to be reprocessed or used for more than one patient and therefore cannot verify that reprocessing can clean and/or sterilize or maintain the structural integrity of the device to ensure patient and/or user safety.

CONTRAINDICATIONS

The contraindications are those which are specific to all endoscopic procedures. The ScopeFlow Air Water and Suction button sets should not be re-sterilized.

STORAGE / PACKAGING

Storage :

In order to prevent any damage, products should be stored in their original packaging, away from sunlight in a cool dry place.

Packaging :

The ScopeFlow Air Water and Suction button sets are supplied in an individual package in a box of 50 units.


PRODUCT DISPOSAL

After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice.

Manufacturer:



Partnership Medical Ltd
Hartley House, Galveston Grove
Fenton, Stoke on Trent, UK

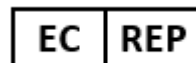

**Advena Ltd, Tower Business Centre,
2nd Flr., Tower Street, Swatar, BKR 4013, Malta**

Instructions for use v5 June 2021

Olympus® is a registered trademark of Olympus Optical Co., Ltd.