















INSTRUCTIONS FOR USE

Clean-Up Bedside Pre-Cleaning Kits : Concentrated detergent for dilution and a sponge within a sealed pouch.

SYMBOL LEGEND :

								
Manufacturer	Do not use if pack is open or damaged	Consult Instructions for Use	EC Rep	Single use	Date of manufacture	Use by date	Product reference number	Lot number
								
Non sterile	CE Mark	Medical device	Not made with Natural Rubber Latex	Not made with DEHP				

PRODUCT CODES AND DESCRIPTIONS

Product Code	Description
82348	Clean-Up Bedside Kits – concentrated enzymatic
82349	Clean Up Bedside Kits - neutral

INTENDED USE

Designed to provide all supplies necessary to begin manually pre-cleaning a flexible endoscope at the patient's bedside, immediately following an endoscopy procedure. Compatible for use with all flexible endoscopes.

INSTRUCTIONS FOR USE

This product must only be used by trained and qualified medical staff or authorized person familiar with endoscope cleaning techniques.

Also refer to endoscope manufacturers written IFU

DIRECTIONS FOR USE

1. The Clean-Up Bedside Kit should be used immediately after the procedure.
2. Tear open the package at the tear notch located at the top of the pouch.
3. Add 500 ml of tap water using the fill-line on the back of the pouch as a guide.
4. Squeeze the cleaning pad inside the pouch to release the detergent and mix into the water. Use the cleaning pad to wipe down the insertion tube.
5. Place the distal end of the endoscope into the Clean-Up Pouch and suction the solution through the suction/biopsy channel as indicated in the endoscope manufacturer's written IFU.
6. Dispose of the Bedside kit as per Hospital guidelines.

WARNINGS AND PRECAUTIONS

For the latest information about endoscope cleaning please consult latest guidelines

Any serious incident that has occurred in relation to this device should be reported to the manufacturer and Competent Authority.

CONTRAINDICATIONS

The Clean-Up Bedside Pouch should not be sterilized for further use.

STORAGE / PACKAGING

Storage :

In order to prevent any damage, products should be stored in their original packaging, away from sunlight in a cool dry place.

Packaging :

The Clean-Up Bedside Pouches are supplied in an individual non sterile package in a box of 100 units.

We recommend that the Clean-Up Pouch is used within 6 hours of opening and diluting.



PRODUCT DISPOSAL


After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice.

Manufacturer:



Partners for Endoscopy Ltd, Hartley House, Galveston Grove
Fenton, Stoke on Trent, Great Britain

Instructions for use v3 May 2024

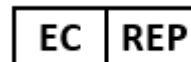

**Advena Ltd, Tower Business Centre,
2nd Flr., Tower Street, Swatar, BKR 4013, Malta**